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GEOTEK® URETERAL STENT (DOUBLE J)**Instruction for Use****ENG**

Ureteral Stent Kit with fully radiopaque Double J or single J stent with a pusher, two clamps and PTFE Guide wire and patient record card. Stents made of Tecoflex PU, there is simple or steerable pusher with the stent. Both end Open and One end Close stent types are indicated at the labels.

Indications: Üreteral stent (DJ Stent) is used for temporary internal drainage from the ureteropelvic junction to the bladder

Contraindications: Serious infections, obstruction, incontinence

Side Effects: Kidney, bladder irritation caused by foreignbody, pain, infection, increased miction-frequency, hematuria

Potential Complications and risks: Disfunction caused by Dislocation, Obstruction and Encrustation of stents. It may cause calculi in the kidney.

Cautions: **Maximum indwelling time is one month.** This Kit should be used by only by physicians experienced in the use of such device.

Regular monitoring of location and free passage of stent is necessary to evaluate stent efficiency. Inflammations of Urinary tract may increase the rate of encrustation and obstruction of the stent.

Ethylene oxide sterilized. Non-pyrogenic. Check the package. if package damaged or opened do not use the products. Check the expiry date and the gauge. Do not use over expired products. Store in dry place, between 5 – 30 °C temperature, protect from sun light. Possible allergic reactions should be considered. **For single Patient Use Only.** Do not attempt to clean or resterilize this product. Disposal as a biohazard material after use.

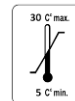
Placement of DJ Stent:**Stent One end Close:**

- Load the rigid end of guide wire in to stent via open end and push it to the very end of closed tip.b. Insert the pusher over guide wire and advance it to the distal end of stent and fix the stent and pusher with clamp. Please keep guide wire taut during this process.
- Insert the stent to the cystoscope channel. When the clamp reaches the opening of channel, remove the clamp.
- Push the stent in to the ureter and renal pelvis.
- When the placement is complete, withdraw the guide wire gently, and place the stent with pusher. Remove the pusher from channel.

Stent Both ends Open :

- Place the guide wire in to Renal pelvis under x-ray control.
- Insert the Stent over guide wire and advance it with help of the pusher to placement area.
- When the stents reach target area, withdraw the guide wire while hold the pusher in current position.
- After stent positioned remove the pusher gently.

Note: These instructions are NOT meant to define or suggest any medical or surgical technique. The individual practitioner is responsible for the proper procedure and techniques to be used with this device.

STERILE EO

Single Use Storage limits Expiry Date Ethylene oxide sterilized

1984 : CE mark and NB identification number

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GEOTEK MEDİKAL VE SAĞLIK HİZ.LTD.ŞTİ