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GEOTEK® URETERAL DILATOR**ENG****Instruction for Use**

It is a dilatation set consist of dilators between 8Fr to 30Fr.

Indications: intended for Dilation renal track/path to Kidney. Made of Medical grade Polyurethane & PP combined to provide little rigidity for stress applied for dilation. This material is mixed with Bismuth / Barium to provide Radio-opacity..

Contraindications: Infections, obstruction, incontinence

Potential Complications and risks: Potential damages of perforation and bleeding.

Cautions: This Kit should be used by only by physicians experienced in the use of such device.

Ethylene oxide sterilized. Non-pyrogenic. Check the package. if package damaged or opened do not use the products. Check the expiry date and the gauge. Do not use over expired products. Store in dry place, between 5 – 30 °C temperature, protect from sun light. Possible allergic reactions should be considered. For single Patient Use Only. Do not attempt to clean or resterilize this product. Disposal as a biohazard material after use

Procedure:

- The kidney is penetrated with proper technic with a puncture needle
- A guidewire is sent through the needle ,with the help of the guide wire ,from 8F through 30 F necessary dilation is performed by using dilators successively
- After the dilatations are provided, a sheath is inserted.
- The operation is initiated by inserting a nephroscope through the sheath .

Note: These instructions are NOT meant to define or suggest any medical or surgical technique. The individual practitioner is responsible for the proper procedure and techniques to be used with this device.

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GEOTEK MEDİKAL VE SAĞLIK HİZ.LTD.ŞTİ