

## MAXICORE REUSABLE BIOPSY GUN

### INSTRUCTION FOR USE:


It is a reusable full automatic biopsy gun used for histological core samples from soft tissue and organs. MAXiCORE Gun full automatic biopsy instrument with 2.2cm advancement.

**Indications:** It is used for getting histological core samples from soft tissue and organs

**Contraindications:** This gun should be used by a physician trained in interventional techniques and familiar with the possible side effects, typical findings, limitations, indications and contraindications of biopsy procedures. Physician judgment is required when considering biopsy on patients with bleeding disorder, or receiving anti-coagulant medications.

**Cautions:** Not Sterilized. Reusable. It must be used only in hospitals by physicians familiar with the device and procedures. It must be cleaned and sterilized after each patient. Check the working mechanism of the biopsy gun after sterilization procedures. Do not clean with bleach or any other chemical reagents. The biopsy gun can be cleaned by enzymatic detergents with warm water. Be sure that it is dried entirely after cleaning. The biopsy gun can be sterilized with ethylene oxide. Before sterilization, the gun must be cleaned carefully.


### Needle Placement and Biopsy Procedure:

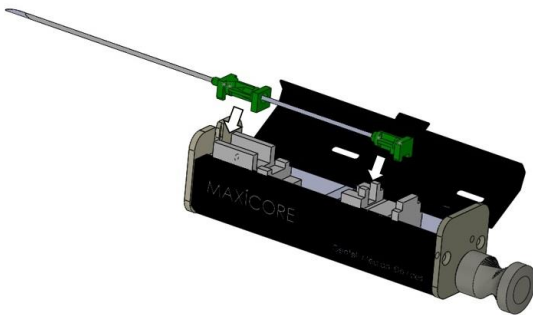

1. Open the lid and place core needle into the gun



2. Pull the charging arm back firmly to cock the gun

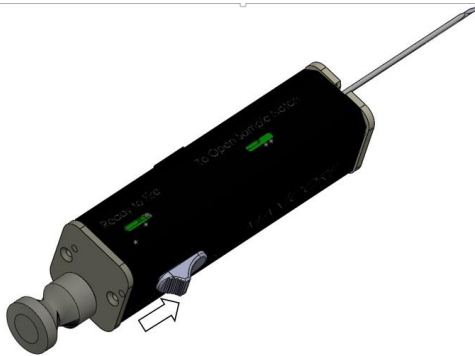

3. Push to trigger to fire



4. Pull the charging arm back half way to expose and fix the specimen notch for removing biopsy specimen

**Note:** These instructions are NOT meant to define or suggest any medical or surgical technique. The individual practitioner is responsible for the proper procedure and techniques to be used with this device.

CE 1984

GEOTEK MEDİKAL VE SAĞLIK HİZ. LTD. STİ  
 İvedik OSB, Ağaç-Metal Sitesi 1436Sk. No:12  
 06378 ANKARA - TURKEY  
 TEL:+90312 3959852 FAX:+90312 3959853